

Improved Performance at about Half the Price of the Competition!



APB™
All Purpose Boot



PQ PegAssist™
Off-loading Insole

The All Purpose Boot and PQ PegAssist™ combination provides a protective closed toe shoe with the excellent off-loading properties of the PQ insole. It's efficient as an off the shelf item, effective and provides great value for the clinician and their patient.


Features and benefits:

- › The All Purpose Boot's Relaxed rocker sole reduces plantar pressure by over 25%.
- › Pressure to the wound site may be reduced by an additional 60% when using the PegAssist™ Off-loading Insole.
- › The PegAssist™ features removable pegs that allows for targeted off-loading.
- › Closed-toe design features a deep toe box.

Note:

Walk on the Poron surface, NOT THE PEGS. The Poron surface of the Peg Assist provides a comfortable support base. Walking directly on Pegs can cause multiple issues such as - the pegs to shift while the patient ambulates, pegs collapse into the off-loaded area, ring edema can form, the wound voids into the off-loaded area.

PQ PegAssist™ Cross Section:

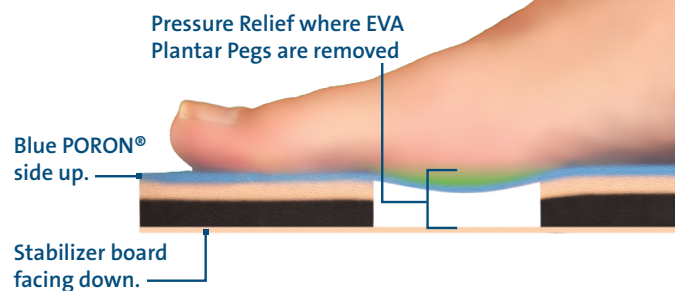

Video demonstration:

Proper assembly and application of the PegAssist™ Off-loading Insole

Scan QR code or visit:

darcointernational.com/pegassist

DARCO